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Caution:

Don't mark on the phantom with pen or leave printed materials contacted on its surface.
Ink marks on the phantom will be irremovable.

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This product is intended to be used for medical training. Do not use for other than the intended purpose.

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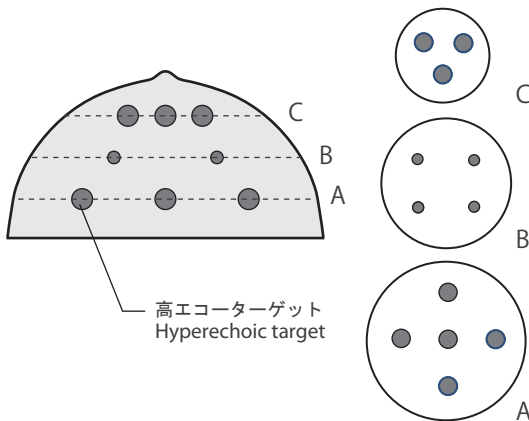
■Specifications

Size of targets: 6 mm dia. and 10mm dia.
Type of targets: Hyperechoic (blue)
Number of targets: 12 pieces
Phantom size: 136mm dia.x70mmH
Phantom weight: 700g

■模擬腫瘍配置図 Targets in the breast phantom

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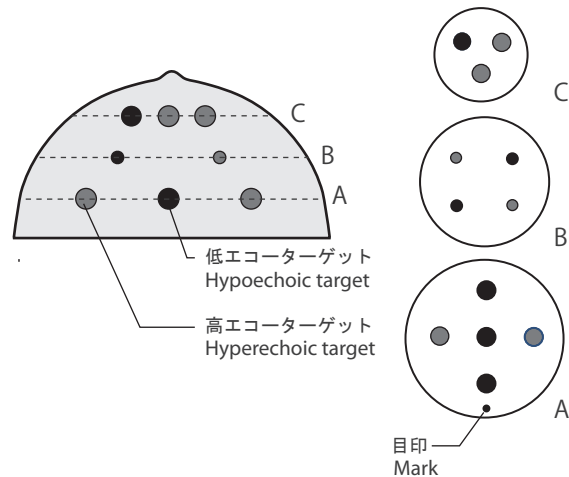
■Specifications

Size of targets: 6 mm dia. and 10mm dia.
Type of targets: Hyperechoic (blue),
Hypoechoic (red)
Number of targets: 12 pieces
Phantom size: 136mm dia.x70mmH
Phantom weight: 700g

■模擬腫瘍配置図 Targets in the breast phantom

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Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:

15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81-75-605-2510 Fax: +81-75-605-2519

Kyotokagaku USA Office: USA and Canada sales and services

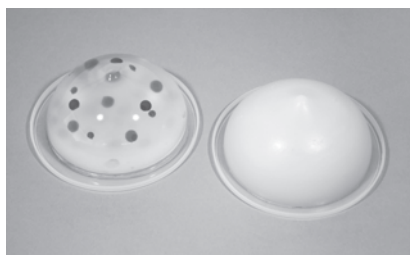
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867

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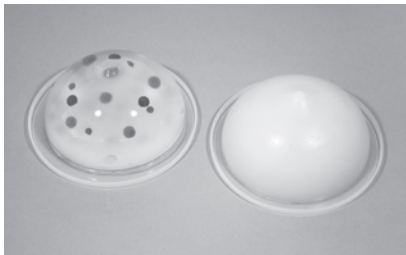
Please read Preparation/ Training

General information Training session

Set includes

- Duo Set: 1387-000
(transparent + opaque type)
- Transparent Set: 11387-100
(1 pair of transparent type)
- Opaque Set: 11387-200
(1 pair opaque type)

11387-000 (transparent + opaque type)

⚠ Caution

- This is a disposable product.
- The aqueous material used in this product contracts as time passes after the package is opened.
- After opening the package, store the product along with the case within a sealed bag or a similar container.
(This method is only for temporary storage. You can delay, but cannot stop contraction and deformation of the product by this storage method.)
- No fluid injection is allowed.
- Do not stretch or press hard on the puncture area.
- Dispose the product as general waste after use.

1、 Befor training

① Remove the back cap on the case.

② Remove the film from the case.

③ Take out the product from the case while inserting a finger between the case and the product to let the air get into between them, then put the product on the back cap.

2、 Training session

① Conduct puncture training under ultrasonic guide after applying water or ultrasonic gel on the product.

※ Temporary storage

After you have removed the transparent film from the case, store the product along with the case after closing the back cap in a sealed bag or similar container. (The product can be stored only for a short while by this method.)